



AnyOne Internal / External / OneStage Surgical Drilling Sequence

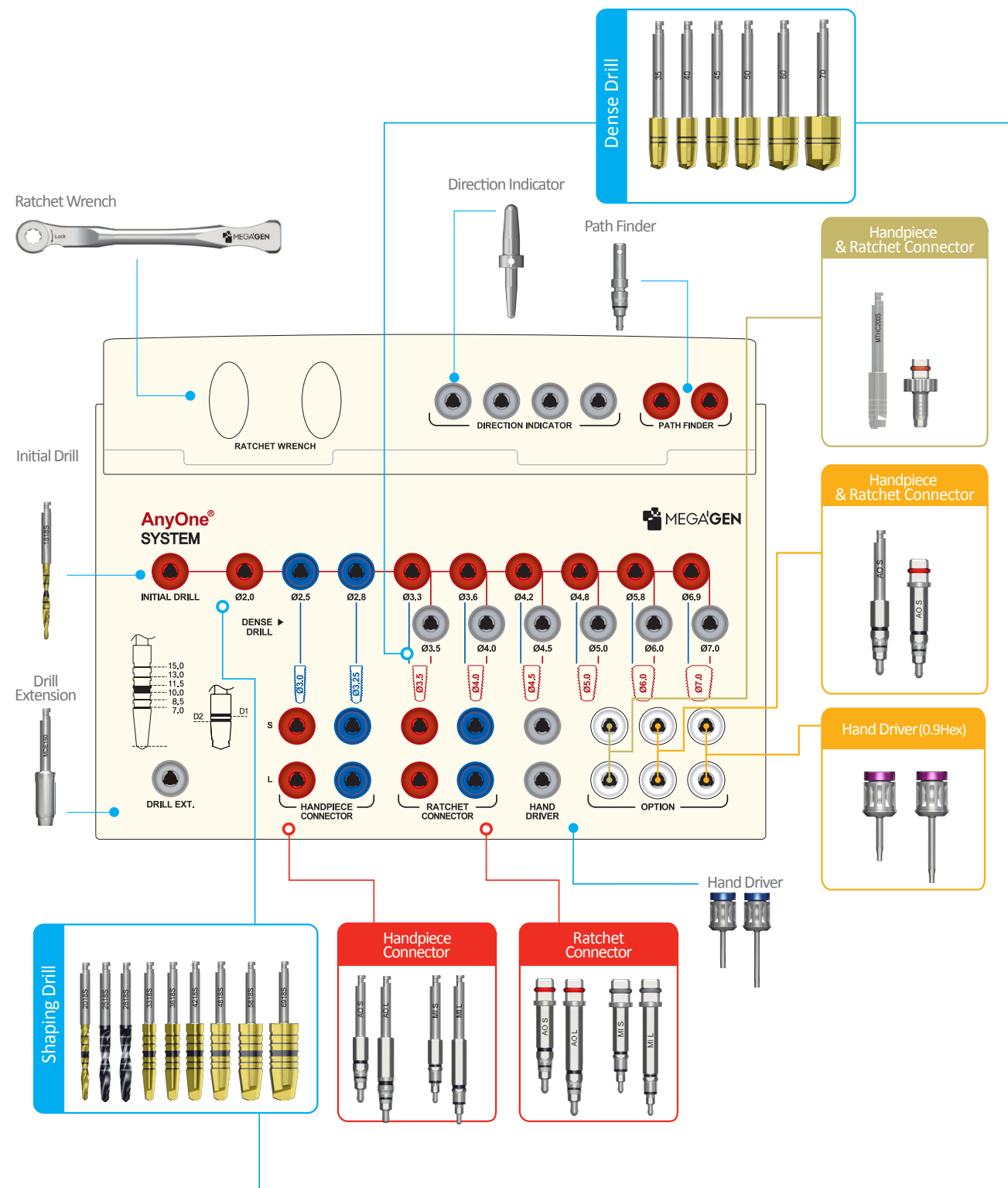

Shaping Drill


- Each drill has depth marking lines from 7.0mm to 15.0mm
- The dual marking system (grooves and laser markings) provides visual and radio graphic depth verification during surgery.

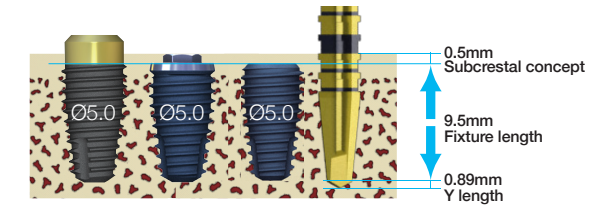

Drill Diameter	Ø2.8	Ø3.3	Ø3.6	Ø4.2	Ø4.8	Ø5.8	Ø6.9
Y length	0.58	0.59	0.68	0.85	0.89	0.94	0.94

※ Actual drill length : Drill length does not normally include the Y dimension of the drill.


※ Markings on the Shaping Drill are 0.5mm longer than the fixture so fixtures will automatically be placed 0.5mm subcrestally if the drilling protocol is followed.

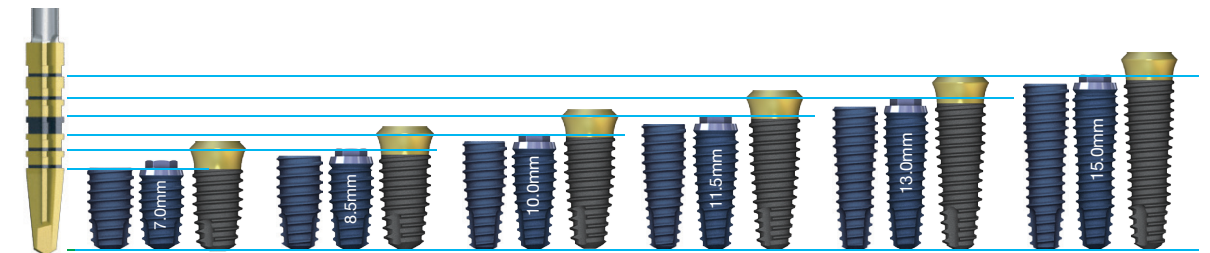

※ To place a Ø5.0 x 10mm length fixture, the required bone depth would be 10.89mm.
For example : 0.5mm(subcrestal concept) + 0.89mm(Y dimension of drill tip) + 9.5mm (fixture length)


Dense Drill

- To control initial stability in dense bone (type I & II), use the Dense Drill to remove and shape the cortical bone.

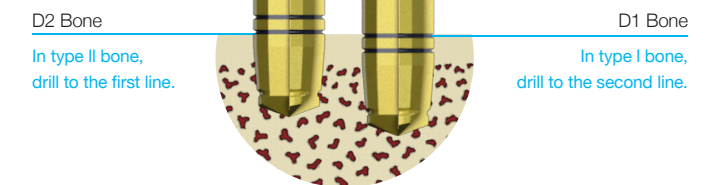

► ► Surgical drilling sequence


- AnyOne fixtures offer optimum initial stability when they are used with the following drill sequence guide, AnyOne implants should be placed 0.5mm subcrestally.

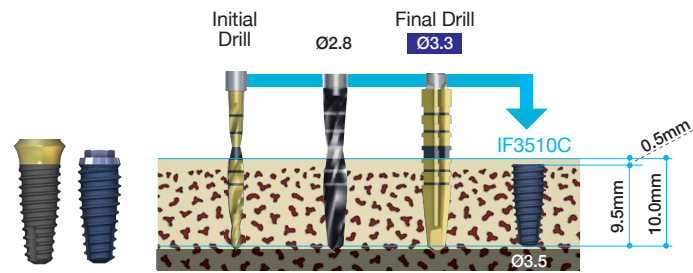

Ø3.5 Fixture

Ø3.5 drilling sequence


10.0mm is the fixture length, The Shaping Drills are 0.59mm longer than the fixture, so total drill depth is 10.59mm.

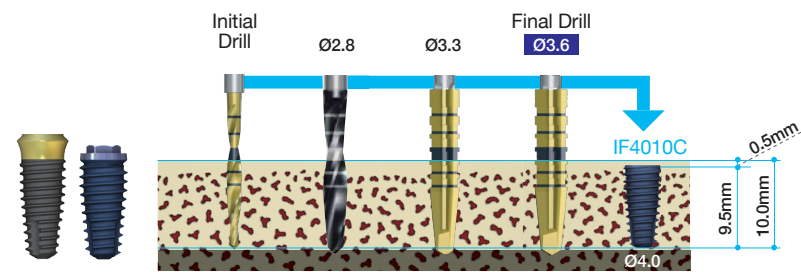

Ø4.0 Fixture

Ø4.0 drilling sequence


10.0mm is the fixture length, The Shaping Drills are 0.68mm longer than the fixture, so total drill depth is 10.68mm.

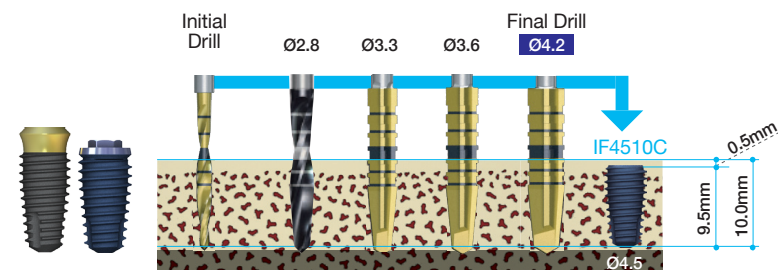

Ø4.5 Fixture

Ø4.5 drilling sequence


10.0mm is the fixture length, The Shaping Drills are 0.85mm longer than the fixture, so total drill depth is 10.85mm.

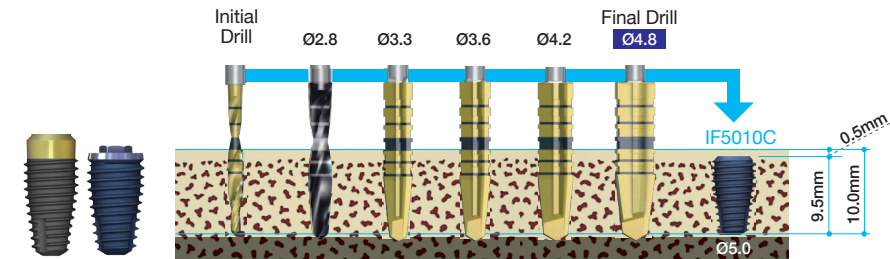

Ø5.0 Fixture

Ø5.0 drilling sequence


10.0mm is the fixture length, The Shaping Drills are 0.89mm longer than the fixture, so total drill depth is 10.89mm.

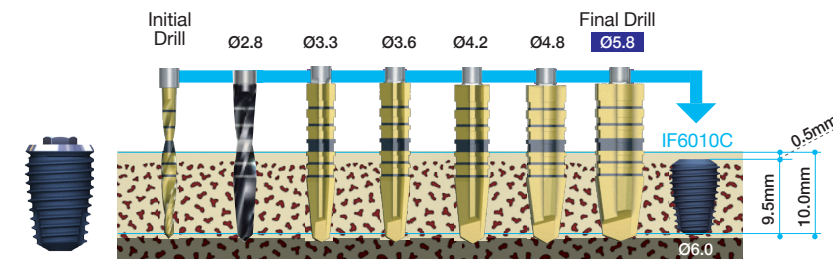

Ø6.0 Fixture

Ø6.0 drilling sequence

10.0mm is the fixture length, The Shaping Drills are 0.94mm longer than the fixture, so total drill depth is 10.94mm.

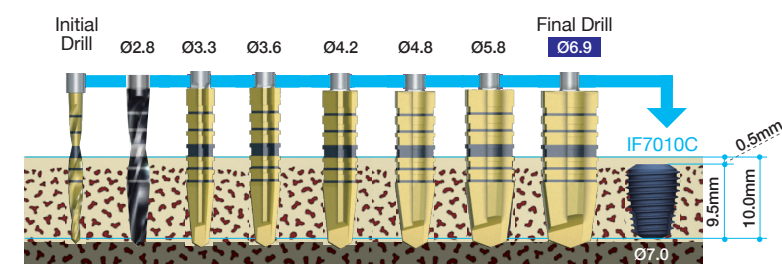


Ø7.0 Fixture

Ø7.0 drilling sequence


10.0mm is the fixture length, The Shaping Drills are 0.94mm longer than the fixture, so total drill depth is 10.94mm.

▶▶ Handpiece & Ratchet Connector

Internal

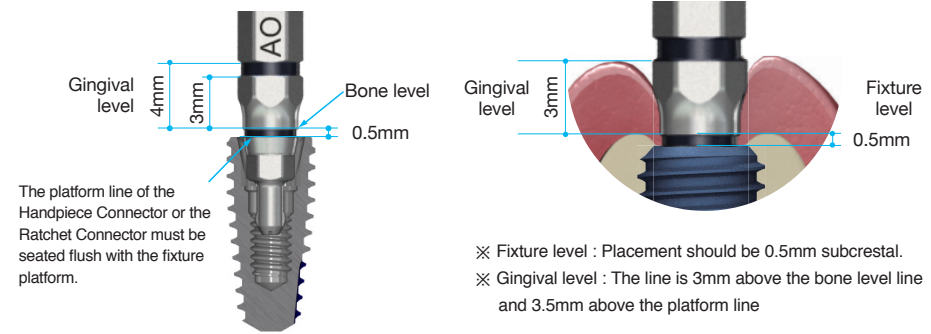

External

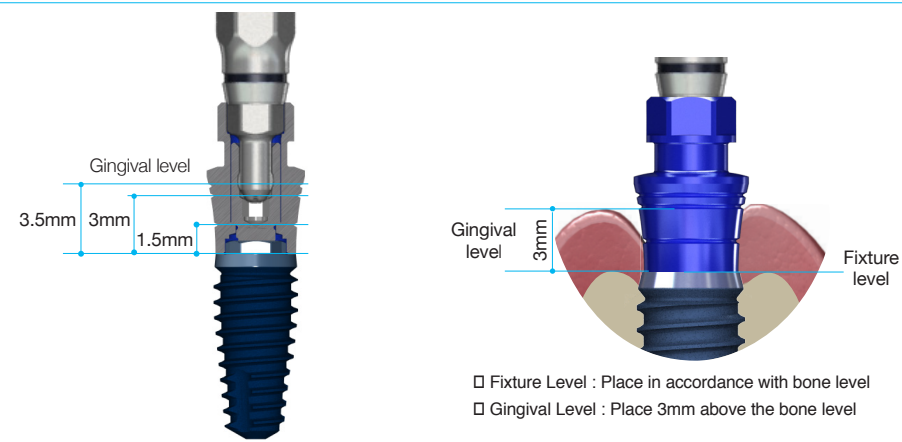

OneStage

